

INQLINE™

IMPORTANT SAFEGUARDS AND WARNINGS

- This is a quick guide which **MUST** first be read and understood before using this securement system.
- **J-Hooks** must be attached to a solid wheelchair frame (no spokes, wheels or movable components).
- **Shoulder and Lap Belt** (we recommend using both) **MUST** go across occupant's shoulder and hip, and not be held away from the occupant's body by wheelchair components.

SECURING WHEELCHAIR


1

1. Wheelchair behind ramp.
2. Lock brakes or power off chair.
3. Press and hold both buttons (4 sec) to release front retractors.




2

4. Attach front J-Hooks to solid frame (30 sec).
5. Unlock brakes and keep chair powered off.
6. Press and hold Z Button.
7. Push Thumb Controller forward—moves chair up ramp.
8. **CAUTION:** Move Controller left or right to adjust chair's direction.

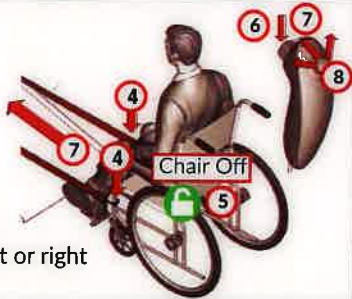


3

9. Release Z Button and place Controller in storage.
10. Attach rear retractors to floor.
11. Attach J-Hooks to solid frame.
12. Press and hold Lock Button (3 sec) to lock retractors.

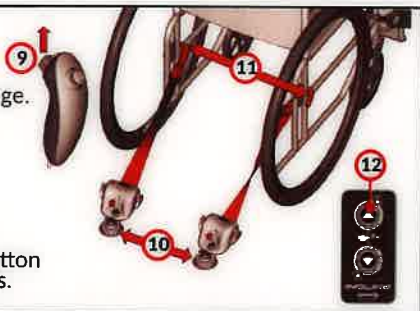

4

13. Buckle shoulder and lap belt in place.
14. Lock brakes and keep chair powered off, ready for transport.

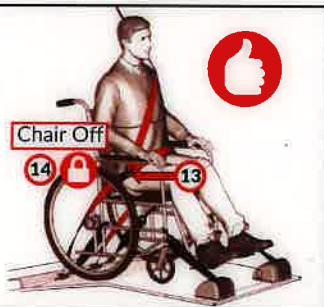

RELEASING WHEELCHAIR


1

1. Unbuckle shoulder and lap belt (place in storage).
2. Unlock brakes and keep chair powered off.




2

3. Press and hold Unlock Button (4 sec) to release retractors.
4. Press lever on retractors.
5. Detach rear J-Hooks.
6. Remove retractors from floor and place in storage.

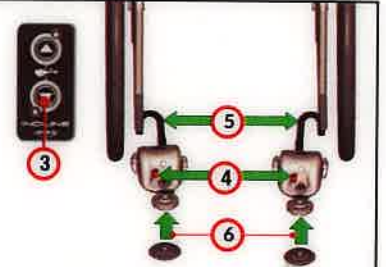


3

7. Press and hold Z Button.
8. Push Thumb Controller downward; moves chair down ramp.
9. **CAUTION:** Move Controller left or right to adjust chair's direction.
10. Release Z Button and place Controller in storage.
11. Lock brakes and keep chair powered off.

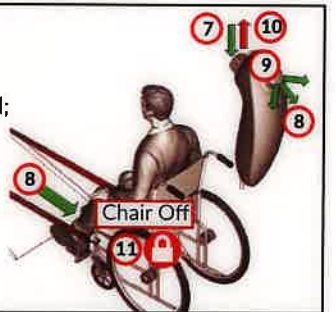

4

12. Press and hold both buttons (4 sec) to release retractors.
13. Return J-Hooks to stowage.
14. Unlock brakes or power on chair.
15. Place chair in secure area.

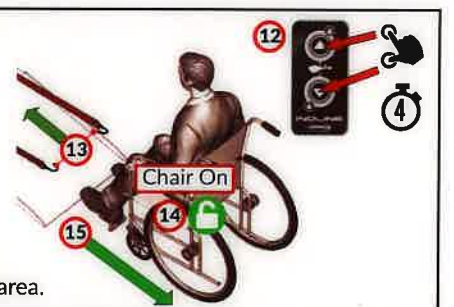

☎ Contact a Q'Strain® Authorized Dealer or Customer Service (800) 987-9987. www.qstrain.com

These instructions contain the most current information at the time of printing. Q'Strain® reserves the right to alter or modify contents and/or components without notice.

© 2017 Q'Strain®. All Rights Reserved.

QC17007 Rev A